

Service
Service
Service



Philips Domestic Appliances and Personal Care

Service Manual

PRODUCT INFORMATION

Blood pressure monitor

Upper arm blood pressure monitor with automatic inflation. The electronically controlled pump inflates the cuff to the pre-pressure value that fits the user. With this fuzzy logic principle, it is not necessary to adjust the pre-pressure by means of additional controls. Shows systolic, diastolic blood pressure and heart rate on the display at the same time.

FEATURES

Battery low indication on display
Memory for the last measured values
Automatic switch off after 1 minute

This blood pressure monitor meets the Medical Device Directive 93/42/EEC for medical appliances.

TECHNICAL DATA

Operating principle	: Oscillometric measurement
Inflation principle	: Automatic electronically controlled (Fuzzy Logic)
Deflation	: Automatic
Power consumption	: more than 300 measurements (Alkaline or Philips Power Life batteries)
Display	: LCD display
Controls	: - On / off / start button - Memory button - Release knob for the protective lid
Measuring range	: Blood pressure : 20 - 280 mmHg Heart rate : 40 - 200 pulses/minute
Operating temperature	: + 10 °C to + 40 °C
Operating humidity	: 30 % to 85 % RH
Accuracy	: Blood pressure : ± 3 mmHg (absolute accuracy tested on manometer) Heart rate : pulse ± 5 %
Battery	: 4 x 1.5 V, size AA (LR06) Alkaline, Philips Power Life or rechargeable batteries

PARTS LIST + EXPLODED VIEW

In connection with the M.D.D. requirements, service is only carried out for the mentioned below components

pos.	description	consisting of	service code
1	cuff large (circumference upper arm > 32 cm.)	tube 2 included	4822 690 10208
	cuff standard (circumference upper arm > 22 < 32 cm.)	tube 2 included	4822 690 10209
	cuff small (circumference upper arm < 22 cm.)	tube 2 included	4822 690 10211
2	tube		4822 530 10531
3	L - shape connector		4822 528 11328
4	pouch		4822 600 10851
5	cover battery compartment	sponge included	4822 442 01804

Direction For Use :	NOR, SWE, FIN, DNK, ESP, PRT	4822 736 16829
Direction For Use :	Arab, IRN, TUR, GBR	4822 736 16831
Direction For Use :	GBR, FRA, DEU, NLD, ITA, GRC	4822 736 16832
Direction For Use :	GBR, S - Chinese, T - Chinese, KOR and Malay-	4822 736 16833

